



Memo to: User of the UniqueBrief®/WellnessBrief®  
 From: First Wellness  
 Re: Application of UniqueBrief®/WellnessBrief®  
 Message: **These instructions are imperative for benefits of the Briefs to take effect.**

## Instructions for use:

- The airflow leakproof cuffs must be loosened and raised between points A & B prior to application. *see picture 1*
- For maximum protection, it is important that the 2 lower tabs be pulled upwards in the direction of the arrows for a snug fit. *see picture 2*
- Upper tabs should also be fastened in the proper direction. *see picture 2*



**Reseal-able Tapes:** Untape the white tab, and leave the blue tab on, then reseal the white tab onto the brief itself. Adjust tab for a comfortable and snug fit. *see picture 3*

**Suggested Use:** 1st change: 9-10PM, 2nd change: in the morning, 3rd Change: only if needed.

**Wetness Indicator:** Change is due when all printing along the central portion of brief disappears.

1st Time Use



2nd Time Reseal

For more product information: . . . . .info@uniquebrief.com  
 Product Sales: . . . . .sales@uniquebrief.com  
 Medical inquiries related to the Briefs: . . .medic@uniquebrief.com

Visit us on the web at:  
[www.uniquebrief.com](http://www.uniquebrief.com)